

# Temptag<sup>®</sup>

## Step one: INSTRUCTIONS

1. Tear a tag from its packing strip and insert pin into hole ensuring that the female cap is on facing side. Gently bend the hinge and ensure that the male point lines up with the female hole.

## Step two: APPLICATION

2. Place thumb and forefinger close to the area the tag will pierce and apply the tag. The point should pierce the ear first and the female part should be closed upon the point until a click is felt. The tag is then locked into position.



3. Where application by hand proves difficult a Dalton U-Tag applicator may be used.

## PRODUCT FEATURES

The Temptag<sup>®</sup> is fitted by hand, no applicator required. Designed for short-term batch marking in the field or at birth. It is an alternative to spray marking.

## COLOUR CODE PINS

Temptags<sup>®</sup> are available in white only but fixing pins can be obtained in a variety of colours for batch identification.

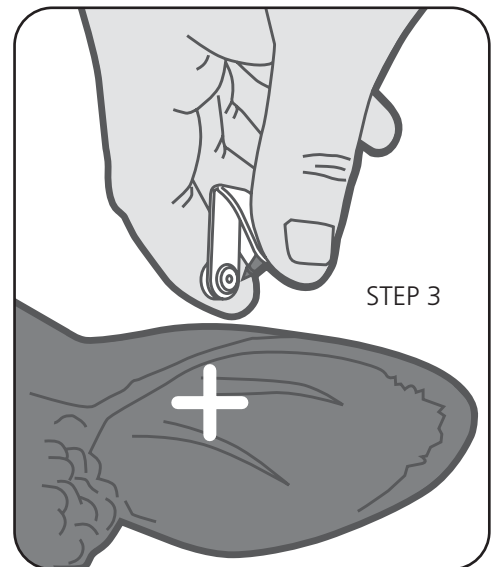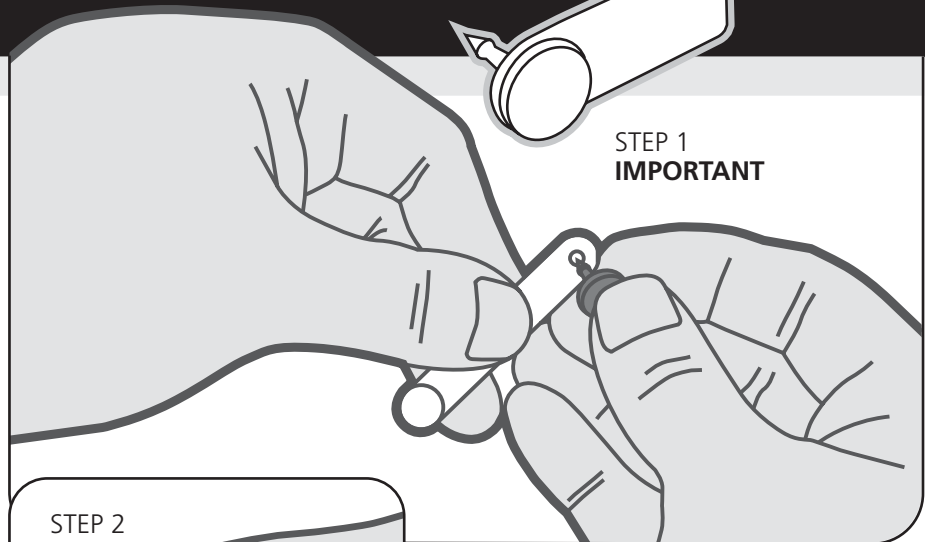

## CUSTOMER SERVICE

In the unlikely event that your tags have been damaged in transit, telephone our customer services dept. immediately.

Where we shall do our utmost to solve the problem. Do not proceed with tagging under any circumstances!

For assistance or a  
free Catalogue or samples call;

**+44 (0) 1491 419 000**

or email: [info@daltonid.com](mailto:info@daltonid.com)